

+ A soft non-adherent dressing with a 'key hole' aperture to allow use with a tracheostomy tube or other drain or stoma.¹⁻²

Smith+Nephew

ALLEVYN[®]
TRACHEOSTOMY
Foam Dressing

Application



- 1 Cleanse the wound in accordance with normal procedures. Prepare and clean the skin surrounding the tracheostomy tube by removing excess moisture.



- 2 Fit the ALLEVYN TRACHEOSTOMY Dressing around the stoma or tracheostomy tube.
- 3 Apply the white, patterned surface of the dressing toward the patient's neck.



- 4 If required, the dressing may be cut to shape.



- 5 Additional fixation, such as OPSITE[®] FLEXIGRID[®] Transparent Film Dressing or HYPAFIX[®] Dressing Retention Sheet may be used to secure the dressing.



- 6 During the early stages of treatment, the tracheostomy site should be inspected frequently. The dressings can be left in place undisturbed for up to 7 days, or until exudate is visible and approaches 3/4" from the edge of the dressing.



- 7 To remove ALLEVYN TRACHEOSTOMY Dressing, remove the retention dressing or bandage and lift the dressing away from the wound.

ALLEVYN TRACHEOSTOMY ordering codes

Size (in)	S+N Code	Each/unit	Unit/case
3 1/2" x 3 1/2"	66027640	10	8

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

References 1. Smith & Nephew 2019. Volunteer evaluation of AWM product features. Internal report EO.AWM.PCSgen.007.v1. 2. Smith & Nephew 2006. A prospective user evaluation on the effect of process changes to the manufacture of ALLEVYN NON-ADHESIVE wound dressings. Internal report CER/CE/022/ANA.

Smith & Nephew, Inc., Fort Worth, TX 76109 USA T 1-800-876-1261 F 727-392-6914

www.smith-nephew.com [®]Trademark of Smith+Nephew. All Trademarks acknowledged. ©2021 Smith+Nephew ALPE14-28657-0321